

Register this test kit here: www.epi-age.de/register



BEWARE: TEST CANNOT BE PROCESSED AS LONG AS YOU HAVEN'T REGISTERED YOUR TEST KIT WITH YOUR SECURE ID



PLEASE DO NOT CONSUME ANY SUBSTANCES CONTAINING COLLAGEN ON THE DAY OF THE SAMPLE COLLECTION.

CAUTION:
30 MINUTES BEFORE SALIVA COLLECTION STRICTLY AVOID EATING, DRINKING, SMOKING TOOTH BRUSHING, CHEWING GUM, ETC.

epiAge Instructions for use

TEST KIT FOR SALIVA COLLECTION

Please take 3-5 minutes for the test

WARNING!

- Do not swallow the preservation fluid
- Avoid eye contact with the fluid
- Keep out of reach of children



1 Carefully unscrew the blue cap from the test tube containing the preservation fluid and screw the funnel onto the tube.

Caution: 30 minutes before saliva collection, strictly avoid eating, drinking, smoking, tooth brushing, chewing gum, etc.



2 Repeatedly spit clean saliva (not mucus) into the funnel, which you have just affixed onto the test tube, until the tube is filled to the arrow marking (5ml).

Please note: Spit patiently several times into the funnel. Take your time inbetween spits.



3 Carefully unscrew the funnel from the tube to avoid spilling the collected saliva.

The funnel can be disposed of.



4 Screw the blue cap back onto the test tube. Turn it upside down and shake it 10-15 times to ensure that the saliva and the preservation fluid are completely blended.



5 Place the test tube into the ziplock bag.



6 Please store this leaflet carefully as it contains the digital secure ID you will require in order to retrieve your test result.



7 Send in the sample using the enclosed shipping slip.

Please follow the instructions of the reseller where you purchased the test.



8 Register the test kit here: www.epi-age.de/register

Without registration, you CANNOT retrieve your test result!

Your digital Secure ID



Should you have any questions regarding your sample or results, please contact us, mentioning your secure ID number.

e-mail: support@epi-age.de
www.epi-age.de/support

Address:
epiAge Deutschland
Jakob-Klar-Straße 4
80796 München
GERMANY

Manufacturer: **HKG epiTHERAPEUTICS Ltd.**
Unit 313-315, 3/F Biotech Centre 2,
11 Science Park West Avenue
Hong Kong Science Park
Shatin, N.T., Hong Kong

© 2021 | epiAge Deutschland | in cooperation with
HKG epiTherapeutics Ltd. All rights reserved.