Instructions for Use

In total, sampling procedure takes **1 hour and 10 minutes** due to 1h of not eating or drinking and 10 min prior to sample collection.

In case of **symptoms of sickness postpone the sampling** until you are back to health.

Perform the **sampling on the same day you intend to send the sample** via post.

Do not eat, drink (except for water), smoke, brush teeth or chew gum for **1 h** before collecting the sample. Rinse your mouth with water and wait **10 min** before a buccal swab collection.

Before collecting the sample, scan the QR code using your mobile phone or enter the link via PC. **Create an account** or **log in** to register the test using the **sample CODE** provided on the swab.

Please fill out the **lifestyle questionnaire**.

Open the **swab** and hold the swab by the handle.

Firmly roll the swab **20 times** against one side of the **inner cheek**. Use the index and middle finger of the other hand to press on the cheek while collecting the swab.

Then, **repeat** sample collection with the same swab on the **other side** and firmly roll the swab **20 times** as well. When taking the sample, make sure that you apply **sufficient pressure** along the respective cheek!

Close the **swab**.

Send the sample on the **same day** via post to the testing laboratory with the provided **return envelope**.

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